



## Quick Start Guide

### Phoenix HD 3 & 4 Wheel

**Please Read Before Operating!**

For technical assistance, call toll free (866) 621-4933 or (800) 544-5155.

- 1.** Remove your scooter from its packaging.
- Loosen the tiller adjustment knob, raise the tiller to the desired angle and retighten the knob.



- 2.** Remove the clear plastic insert from the area where the battery pack connects on the front shroud.



- 3.** Set the battery pack onto the scooter directly in front of the seat post. Lock the battery pack in place by rotating the dial on the pack to the locked position.




- 4.** While lifting up on the seat release lever, place the seat onto the seat post. Release the lever and rotate the seat until it locks into place.



- 5.** Insert the armrests into the attachment points in the seat frame so that the armrests face forward.



- 6.** Secure the armrests in place with the provided star knobs at the desired width.

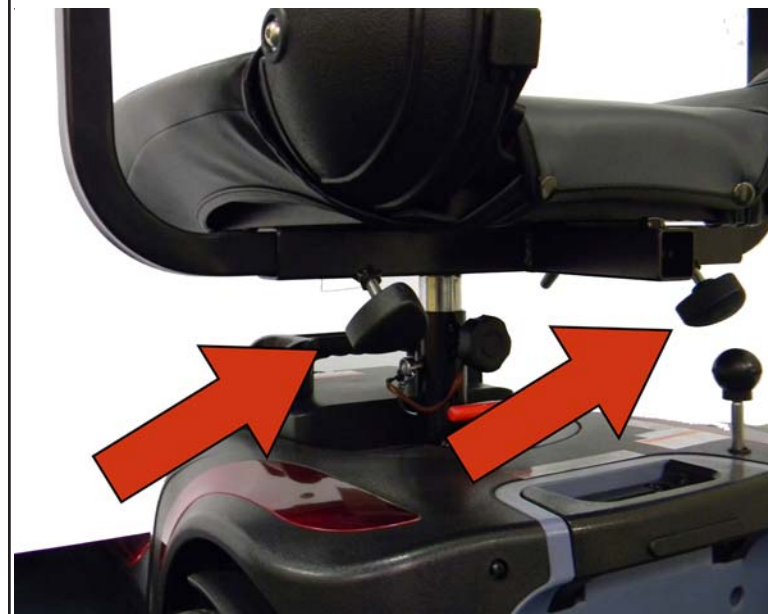

- 7.** Attach the front basket by aligning the bracket on the tiller with the mounting holes on the basket. Then push down to secure in place.



- 8.** Fully charge the scooter's batteries by inserting the round charger connector into either the charging terminal on the scooter's battery pack or the terminal located on the scooter's tiller. Then insert the 3-pronged plug into a standard electrical outlet. Be sure to charge the scooter for 8-10 hours before first time use.



9. Before use, make sure that the scooter's freewheel mode knob is pushed down in the "drive" position.



For information on the comfort adjustments available on your scooter, please consult your owner's manual.

Your Phoenix HD Scooter is designed to be easily disassembled into 5 pieces for convenient transport or storage.

The pieces are:

- Front Section (heaviest piece, 40 lbs.)
- Rear Section
- Battery Pack
- Seat
- Front Basket

To disassemble your scooter:

1. Turn key switch to "OFF."
2. Engage front wheel lock.
3. Place the freewheel mode lever in the "Drive" position.
4. Pull upward on the swivel-release lever to remove the seat from seat post.
5. Pull up on the battery pack to remove it from the scooter.
6. Pull up on the release lever to separate the front and rear sections.



# Phoenix HD Quick Start Guide



## READ BEFORE UNPACKING

This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.

**NOTE:**The illustrations provided may not reflect your actual unit.