EN Side-folding Rollator

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EN

READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR ROLLATOR:

- Consult your doctor, therapist or other qualified professional to determine if this Airgo® Rollator will suit your particular needs.
- IMPORTANT - Before using, make sure that all parts are secure and the Airgo® Rollator frame is fully opened and in locked position.
- For proper positioning and use of your Rollator, imagine a line drawn across the back of the Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk.
- To slow down, squeeze the Brake Handles up. To lock the Brakes, push down on the Brake Handles until the Brakes click into locked position. If the Rear Wheels of the Rollator move while the Brakes are in locked position, DO NOT use the Rollator. Please see adjustment instructions in this manual or call our toll-free number (1-800-363-2381) for return or repair. To release the Brakes from locked position, pull up on the Brake Handles.
- IMPORTANT - Before using the Seat portion of your Rollator, ALWAYS lock the brakes FIRST. DO NOT navigate the Rollator while sitting on the Seat. The Brakes should ALWAYS be locked when the Seat is being used. Risk of fall and serious injury may result if Brakes are not locked while the user is sitting on the Seat. Falls are possible if the user leans or slides backward on the Seat without first installing the Backrest. NEVER attempt to sit on the Seat when the Rollator is on an incline. Serious risk of falls and injury may result. DO NOT push the Rollator while someone is sitting in the Seat. Damage to the Rollator frame or Wheels, and serious injury to the user, could result from improper use. CAUTION - DO NOT exceed the following weight limits, or serious injury could result: 700-917, 700-920, 700-921, 700-922, 700-923, 700-924, 700-926, 700-927 Seat: 300 lbs. 700-928, 700-929 Seat: 400 lbs. The Seat is not intended to transport people or objects. DO NOT use the Seat to carry or move anything.
- Get assistance if you are uncomfortable folding or unfolding your Airgo® Rollator. DO NOT use your Rollator while it is folded. The stability of any Rollator is compromised when its frame is in the folded position, increasing the likelihood of a fall and as a result serious injury to the user may occur.
- Your Airgo® Rollator should be checked periodically to ensure the Brakes are functioning properly and that all nuts and bolts are secure.

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2

EN CHECK ALL PARTS

Carefully examine all parts for shipping damage. DO NOT use the Rollator if there is apparent damage or if there any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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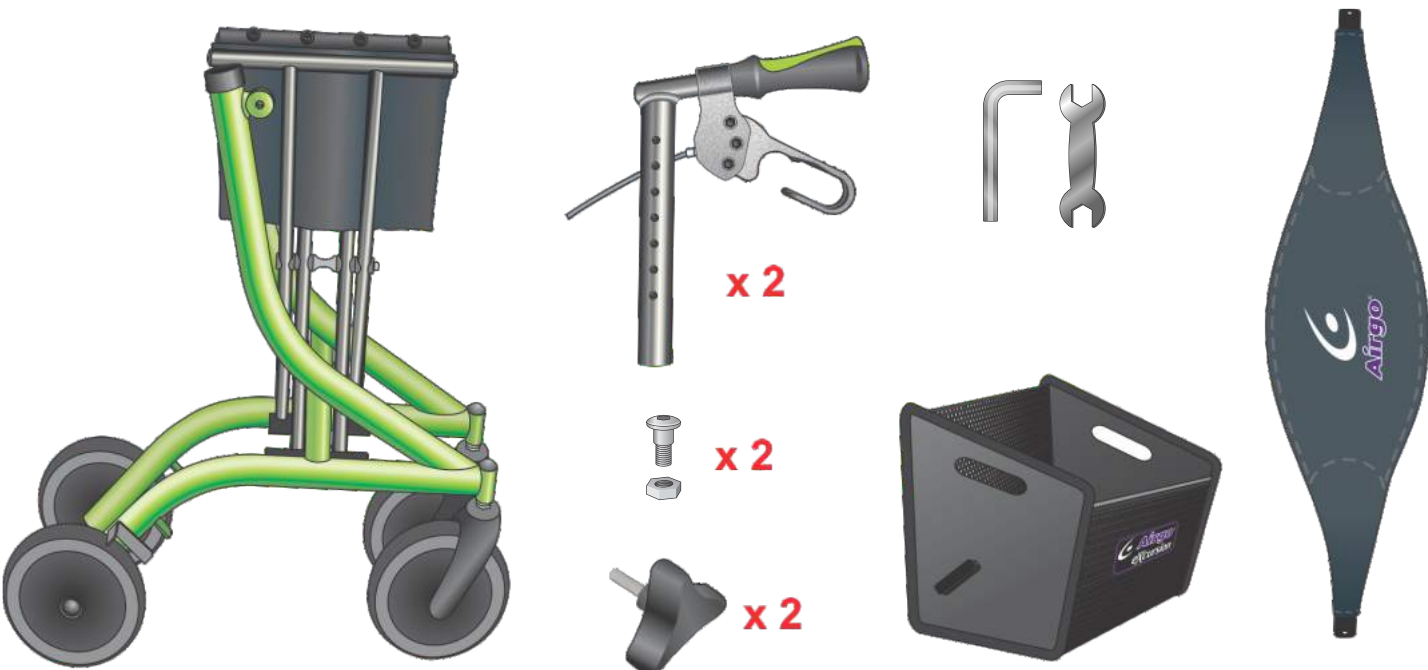
3

EN UNFOLD THE ROLLATOR FRAME

Place the Rollator frame standing upright on level ground. Ensure all 4 wheels are touching the ground. Push down on the Seat Supports (J) to expand and open the Rollator frame. You should hear the Safety Lock (M) click into place when the Rollator is fully unfolded.

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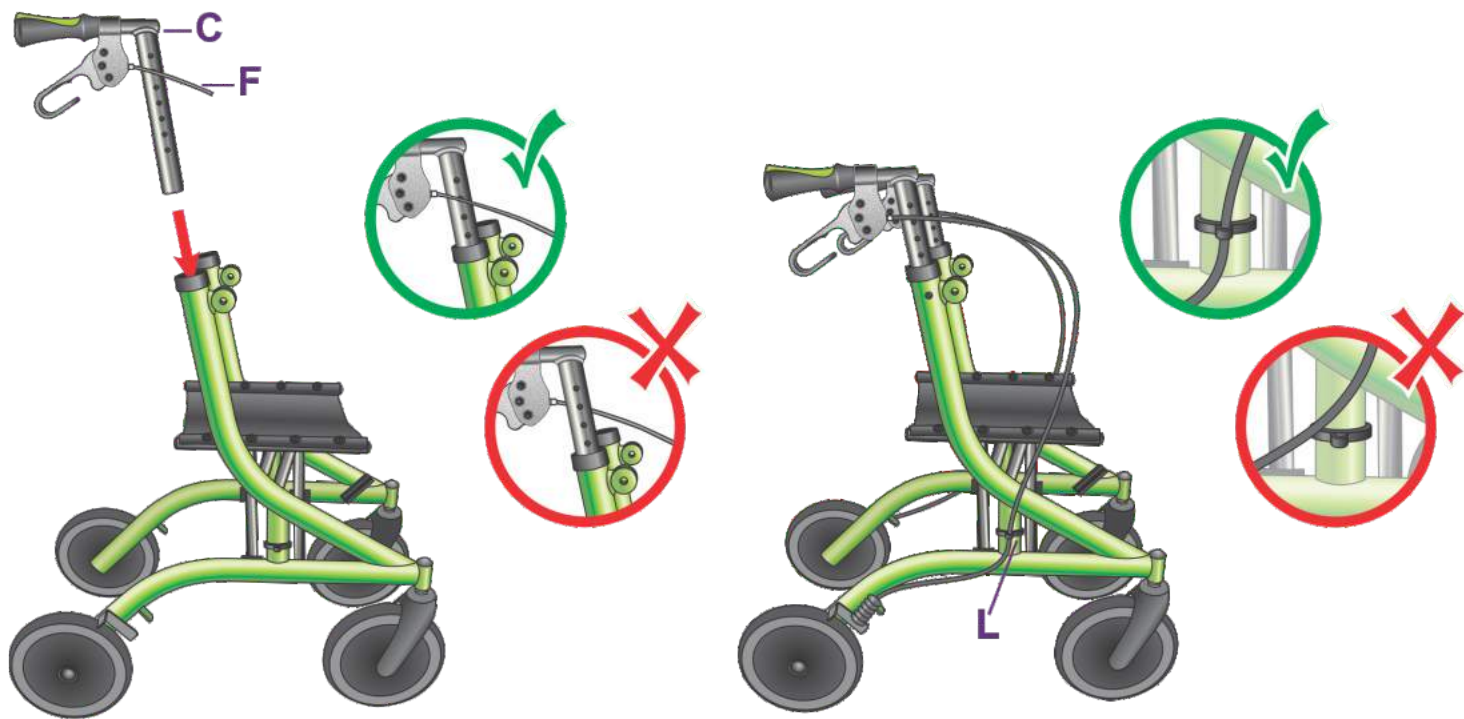
4

EN INSERT THE HANDLEBARS

Insert the Handlebars (C) into the openings of the frame, as shown. Ensure the Brake Cables (F) are on the outside of the handle and frame. Make sure the Brake Cable (F) passes through the Brake Cable Guide (L).

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EN LIMITED LIFETIME WARRANTY

AMG Medical Inc. covers this Airgo® rollator by a Lifetime Warranty on the Frame and a Five (5) Year Warranty on Brake Handles against defects in materials and workmanship. All other parts and components are warranted for a period of 90 days. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:

1. Return your rollator to the Vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-800-363-2381 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of purchase and serial number (on crossbar underneath the seat). Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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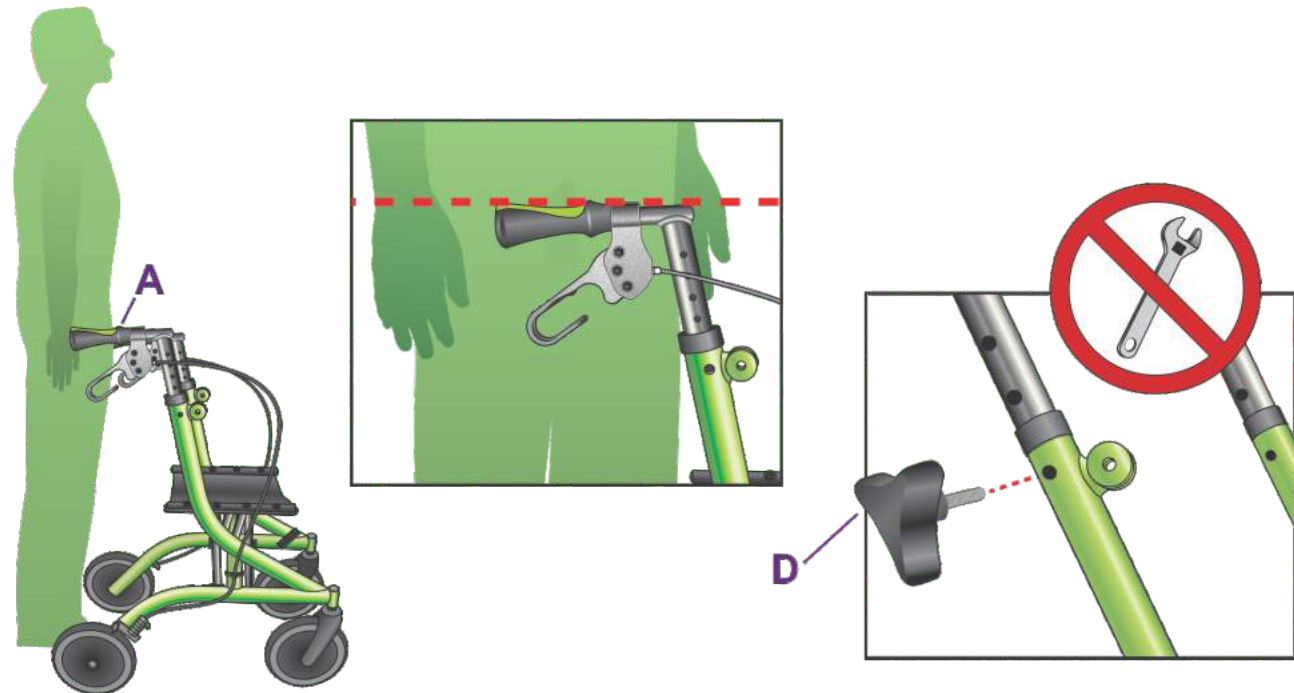
5

EN ADJUST THE HANDLEBARS

Stand upright behind the Rollator, shoulders relaxed, arms hanging at your side, and eyes looking straight ahead. In this position, adjust the Handgrips (A) of the Rollator to a height even with the wrists. Secure the Handlebars at the closest hole to the desired height. Insert and tighten the triangular Hand Screws (D) until secure. HAND TIGHTEN ONLY.

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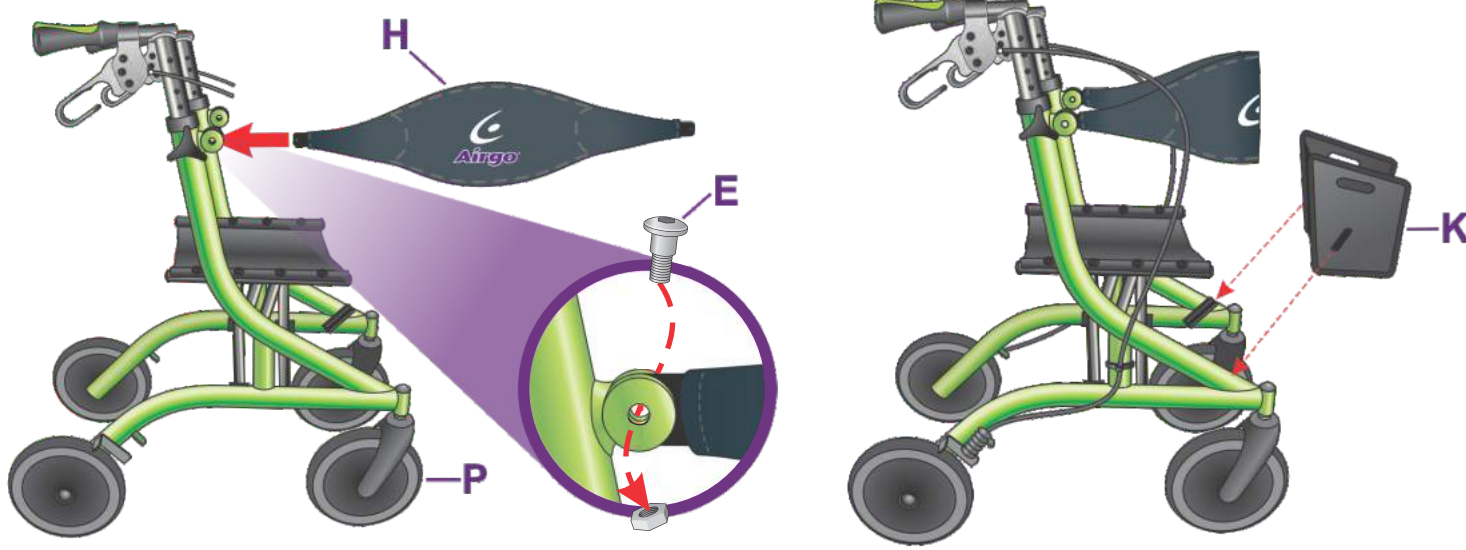
6

EN INSTALL BACKREST / BASKET

Insert the Backrest (H) so that the Airgo® logo faces towards the Front Wheels (P). Slide one end of the Backrest into the fan bracket located near the hand screw. Slide the Backrest Screw (E) from inside of the unit through the hole in the fan bracket, then secure it with the nut and tighten using the allen key and wrench provided. Repeat on the other side. Install the Basket (K) by sliding the Basket Brackets into the frame's Basket Mounts.

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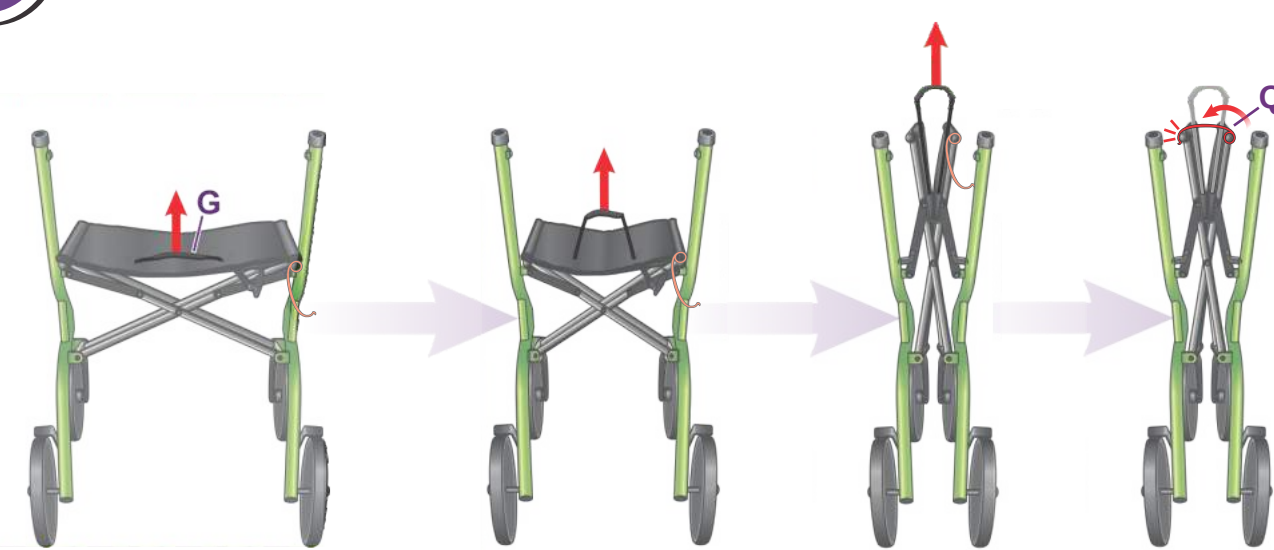
7

EN FOLDING

Before folding the Airgo® eXursion, lock the Brakes by pushing down on the Brake Handles until you feel or hear the Brakes click into locked position. Pull the Folding Strap (G) located on the Seat to release the Safety Lock and initiate folding. The position of the front wheels should be aligned to allow the Rollator to fold completely. Use the Folding Clip (Q) to lock the rollator in the folded position for safer lifting.

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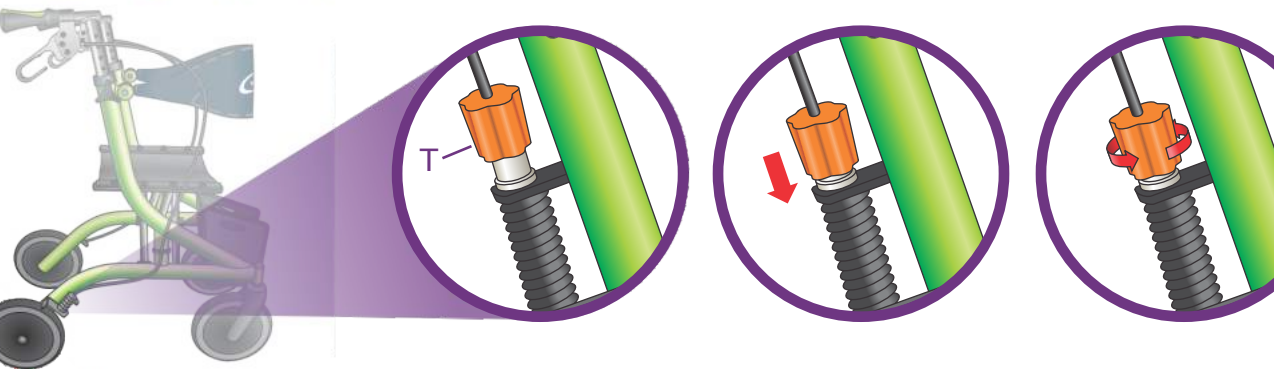
!

EN PERIODIC BRAKE ADJUSTMENT

To adjust the brakes, simply push down the orange brake adjustment knob (T) and rotate. Rotating the adjustment knob clockwise will loosen the brakes while rotating it counter-clockwise will tighten them.

IMPORTANT – Before using the Rollator again, ensure the brakes are working properly. DO NOT use the rollator if it moves while the brakes are in the locked position. If you are unsure of how to adjust your brakes please see your authorized Airgo dealer or call our toll-free number (1-800-363-2381) for help.

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